

We look at healing differently. We're inspired by a passion to help people live healthier, happier lives.

KINESIO® TEX CLASSIC ORIGINS : Using his knowledge of natural healing modalities, Dr. Kenzo Kase® spent several years of research and development, trial, and error to develop the ORIGINAL Kinesio Tape. Debuting in the early 1970s, Kinesio Tex Classic ORIGINAL was the first elastic therapeutic tape, and it revolutionized the medical & sports industries around the world.



Learn Kinesio Taping® with Kinesio University® In-Person and Online Courses. TapeWithWisdom.com/EDU



HYPO-ALLERGENIC



LATEX-FREE



MULTIPLE DAY WEAR



#1 DOCTOR RECOMMENDED BRAND



WATER-RESISTANT



MEDICAL DEVICE

KINESIO TEX
CLASSIC

ELASTIC THERAPEUTIC TAPE

CLINICALLY PROVEN

KinesioTape.com
Facebook: /KinesioTapeOfficial
Twitter: @KinesioTape
Instagram: @KinesioTape
#KinesioTape
#TapeWithWisdom

Elastic Therapeutic Tape
Specially Designed for
CERTIFIED
KINESIO TAPING®

IFU

WATCH HOW TO VIDEOS

KinesioTape.com/Instructions



KINESIO HOLDING CORPORATION

4001 Masthead ST NE,
Albuquerque, NM 87109
855.488.TAPE(8273)
www.KinesioTape.com

Kinesio Importers/Distributors:
tapewithwisdom.com/partners

IFU in additional languages:
tapewithwisdom.com/ifu



EMERGO EUROPE Phone: +31.70.345.8570
Prinsengracht 20 2514 AP The Hague Fax: +31.70.346.7299
The Netherlands
[CH REP] MedEnvooy, Switzerland, Gotthardstrasse 28
6302 Zug, Switzerland



Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

©2023 Kinesio Holding Corporation.
All Rights Reserved.

Original



DYNAMIC • VERSATILITY •
SUPPORT • CORRECTIVE

The Original from Dr. Kenzo Kase® since the early 1970s. Since then, Kinesio® has become the standard and the most trusted name in elastic therapeutic taping.

Kinesio® Tex Classic ORIGINAL: Classic Original is used to provide support and is perfect for deep stimulation to target muscles, ligaments, and joint corrections.

STRONG



SUPER STICK ADHESIVE •
DURABILITY • SECURE HOLD

Kinesio® Tex Classic STRONG: Classic Strong is used to provide stability and is developed with a super stick adhesive for a long-lasting durable hold.

Please use this tape with caution. Do Not Use for pediatric or geriatric populations or anyone with skin sensitives, irritation, tape allergies or comprised skin.

MOVE



ENGINEERD DESIGN •
FLEXIBILITY • COMFORTABLE
• JOINT STABILITY

Kinesio® Tex Classic MOVE: Classic Move features dual threads in a unique wave that moves with you and is used to provide comfortable flexibility, gentle stimulation and joint stability.

TO CARE FOR YOUR APPLICATION:



Rub to activate adhesive.



Use caution.
Clothing can catch edges of tape.



Wait 35 - 45 minutes
before vigorous activity.



Pat or blot
to dry.
Do not wipe.



Direct heat can cause
over-adhering and can
make removal difficult.

• Assistance is recommended to apply applications. • Ensure skin is clean, dry and free of any oil or lotions. • It is normal for wrinkles (convolutions) to appear in the tape when it is applied to the skin and the muscles are relaxed.

PRECAUTIONS: • Consult your physician prior to use. • If you are diabetic, pregnant, have poor circulation, nerve damage, a skin condition or allergy to adhesives, seek medical advice before use. • Seek medical advice if the condition is severe or does not improve within 5 days. • Do not use on open wounds, broken or irritated skin. • Test small area of skin for sensitivity. • A test patch is highly recommended for those with high allergic tendencies. • Stop usage immediately if skin sensitivity occurs. • Use care when applying tension to the tape. • Elongate the target area prior to applying tape. • Remove any excess oils, lotions, sweat, and/or dirt prior to applying the tape. • Store in a cool dry place. Avoid storing product in direct sunlight and/or high temperatures. • Keep out of reach of children. • Do not use hair dryer to dry tape. • Do not remove in the shower or bath. • For easier removal, soak tape with a body oil, such as baby oil, massage oil or similar. • Use by expiry date shown on box.