

We look at healing differently. We're inspired by a passion to help people live healthier, happier lives.

KINESIO® TEX CLASSIC ORIGINS : Using his knowledge of natural healing modalities, Dr. Kenzo Kase® spent several years of research and development, trial, and error to develop the ORIGINAL Kinesio Tape. Debuting in the early 1970s, Kinesio Tex Classic ORIGINAL was the first elastic therapeutic tape, and it revolutionized the medical & sports industries around the world.



Learn Kinesio Taping® with Kinesio University® In-Person and Online Courses. TapeWithWisdom.com/EDU



HYPO-ALLERGENIC



LATEX-FREE



#1 DOCTOR RECOMMENDED BRAND



WATER-RESISTANT



MEDICAL DEVICE

KINESIO® TEX GOLD

CLINICALLY PROVEN

ELASTIC THERAPEUTIC TAPE

KinesioTape.com
 Facebook: /KinesioTapeOfficial
 Instagram: @KinesioTape
 Twitter: @KinesioTape
 #KinesioTape
 #TapeWithWisdom

Elastic Therapeutic Tape
 Specially Designed for

CERTIFIED KINESIO TAPING®

IFU

WATCH HOW TO VIDEOS

KINESIO HOLDING CORPORATION

4001 Masthead ST NE,
 Albuquerque, NM 87109
 855.488.TAPE(8273)
www.KinesioTape.com

Kinesio Importers/Distributors:
tapewithwisdom.com/partners

**IFU in additional languages:
tapewithwisdom.com/ifu**

KinesioTape.com/Instructions



EMERGO EUROPE Phone: +31.70.345.8570
 Prinsessegracht 20
 2514 AP The Hague Fax: +31.70.346.7299
 The Netherlands

UK Responsible Person Emergo Consulting (UK) Limited c/o Cr 360 - UL International
 Compass House, Vision Park Histon Cambridge CB24 9BZ United Kingdom
 CH [REP] MedEnvooy, Switzerland, Gotthardstrasse 28
 6302 Zug, Switzerland



Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

©2023 Kinesio Holding Corporation.
 All Rights Reserved.

Fingerprint



REHABILITATION • CELLULAR
• EDEMA • DISCOMFORT
MANAGEMENT




MULTIPLE DAY WEAR

The Original from Dr. Kenzo Kase® since the early 1970s. Since then, Kinesio® has become the **GOLD** standard and the most trusted name in elastic therapeutic taping.

Kinesio® Tex Gold FP: Incredibly lightweight advanced patented Fingerprint (FP) technology with focused Nano Touch stimulation to mimic a gentle human touch. Kinesio Tex Gold FP Micro-Grip deep set adhesive is made for improved grip and hold yet allows readjustment of the tape without losing its stickiness.

LIGHT
TOUCH+

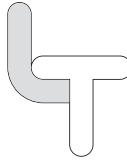

SENSITIVE SKIN •
GERIATRIC • PEDIATRIC
• FASCIAL



SINGLE DAY WEAR

Kinesio® Tex Gold Light Touch+: Exclusive formula designed for sensitive skin populations, including pediatrics, geriatrics, facial applications, and delicate areas of the body. Micro-Air-Pocket patented technology provides a lightweight, gentle touch that delivers a smooth, responsive recoil, while shifting misaligned tissue into its correct position, similar to memory foam.

TO CARE FOR YOUR APPLICATION:



Rub to activate adhesive.



Use caution.
Clothing can catch edges of tape.



Wait 35 - 45 minutes before vigorous activity.



Pat or blot to dry.
Do not wipe.



Direct heat can cause over-adhering and can make removal difficult.

• Assistance is recommended to apply applications. • Ensure skin is clean, dry and free of any oil or lotions. • It is normal for wrinkles (convolutions) to appear in the tape when it is applied to the skin and the muscles are relaxed.

PRECAUTIONS: • Consult your physician prior to use. • If you are diabetic, pregnant, have poor circulation, nerve damage, a skin condition or allergy to adhesives, seek medical advice before use. • Seek medical advice if the condition is severe or does not improve within 5 days. • Do not use on open wounds, broken or irritated skin. • Test small area of skin for sensitivity. • A test patch is highly recommended for those with high allergic tendencies. • Stop usage immediately if skin sensitivity occurs. • Use care when applying tension to the tape. • Elongate the target area prior to applying tape. • Remove any excess oils, lotions, sweat, and/or dirt prior to applying the tape. • Store in a cool dry place. Avoid storing product in direct sunlight and/or high temperatures. • Keep out of reach of children. • Do not use hair dryer to dry tape. • Do not remove in the shower or bath. • For easier removal, soak tape with a body oil, such as baby oil, massage oil or similar. • Use by expiry date shown on box.