



Product Safety Information Sheet

Product Name: Kinesio® Tex Classic STRONG

Date of product launch: 2023

This tape has a super sticky adhesive.

Please use this tape with caution. Do not use this tape if you have a history of any kind of skin irritation.

For pediatric or geriatric populations, we do not recommend using this tape. Skin tears, resulting in blistering, if tape is applied with excessive tension.

Removal of Tape.

Remove the tape with care. Do not remove the tape too quickly. It is more comfortable to gently remove the tape in the direction of hair growth.

The tape can be gently rolled off the skin slowly using the base of the hand. Use the fingers of the other hand to support the skin and reduce discomfort as the tape is being removed.

Tape removal is generally easier if the tape is moist.

Skin preparation is important. The skin should be free of oils and lotions and should be cleaned prior to application.

Be aware of swelling and or edema in the area before and after taping.

Use with care! Do not use this tape over open wounds, fragile skin, poor skin integrity, abrasions, or if you have any tape allergies.